



## STEP 1: ALIGN MARKERS



Align the 2 black horizontal markers that allows to confirm insertion axis


## STEP 2: PREP SYRINGE



Pull the piston and fill the syringe with 20 to 25 cc of air depending on syringe volume

Note : only Luer Lock syringe are compatible with CAVI-T

## STEP 3: PREP CAVI-T


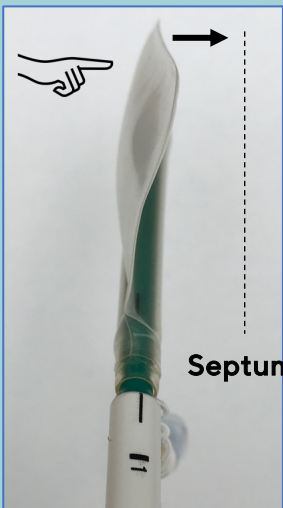


Pre-shape the balloon film with your finger so that it forms a wave oriented towards the septum

Note : in order to facilitate gliding upon insertiion,, CAVI-T can be soaked into saline or sprayed with silicone oil prior to use


## STEP 4: INSERT BALLOON

### Controlled insertion

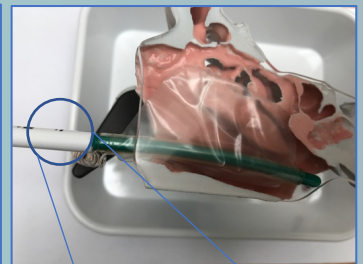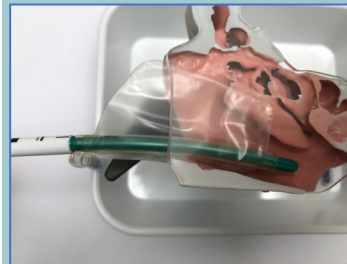



Insert CAVI-T in the nasal cavity while controlling the depth of introduction thanks to distance markers located on the white stiff portion of the deployment guide

## STEP 5: CHECK MARKERS

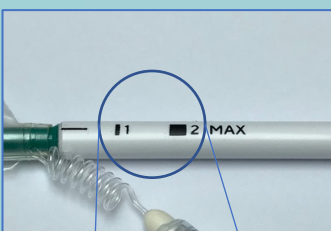


Markers « | 1 » and « ■ 2 MAX » are located respectively 1 and 2 cm away from the proximal portion of the balloon

## STEP 6: PRE-INFLATE

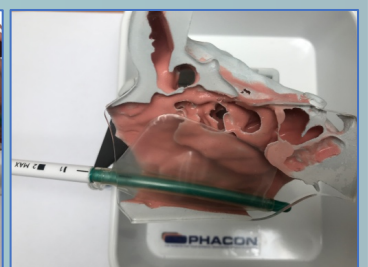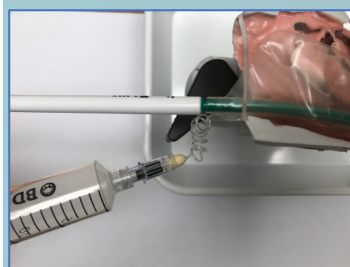

Connect Luer Lock syringe and pre-inflate (approximately 10 cc) to deploy the balloon while maintaining the guide in place. Once the balloon is in proper position, remove the guide.

### STEP 7 : ADJUST POSITIONING

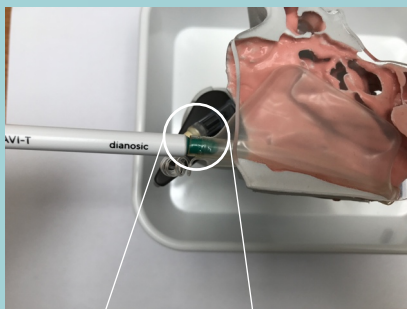

The green, soft proximal tip on the deployment guide can be inserted in the breathing channel to fine-tune product positioning prior to inflation completion


### STEP 8 : FINALIZE INFLATION AND REMOVE GUIDE



Resume inflation while controlling pain and bleeding, without going above 25 cc. Disconnect the syringe and tape the spiraled tubing on the patient's cheek.

### STEP 9 : CAVI-T REMOVAL

After complete deflation of the balloon with a syringe, gently pull on the breathing tube or the spiraled tubing to remove it from the nasal cavity

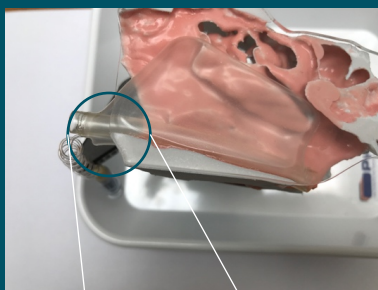

#### Product information:

For any information regarding dianosic products and reporting of adverse events, please contact:

DIANOSIC SAS  
8 place de l'hôpital  
67000 STRASBOURG

Tel : 06 48 03 53 08  
Mail : [contact@dianosic.com](mailto:contact@dianosic.com)  
Web : [www.dianosic.com](http://www.dianosic.com)

