



Stammberger Sinu-Foam[®] Dissolvable Sinus Dressing



Dissolvable Stammberger Sinu-Foam



High Viscosity Sinu-Foam

CMC: A Carboxymethylcellulose Dressing That...

Provides Patient Postoperative Benefits

- Prevents adhesions within the nasal cavities
- Controls minor bleeding and minimizes edema
- Eliminates the need for painful post-op removal of packing

Facilitates Surgical Procedures

- Viscous foam conforms to nasal cavities while providing a moist physical barrier
- CMC remains in place and progressively dissolves with normal outflow
- Remaining foam is easily debrided using gentle suction

Catalog No.	Nasal/Sinus Solutions	Units/Box
RR 650	Stammberger Sinu-Foam Sinus Dressing	2




Sinu-Foam Kit Components


Stammerberger Mixing Steps



1 Draw sterile water or Lactated Ringers solution up to the black line (~8ml) of the small syringe.

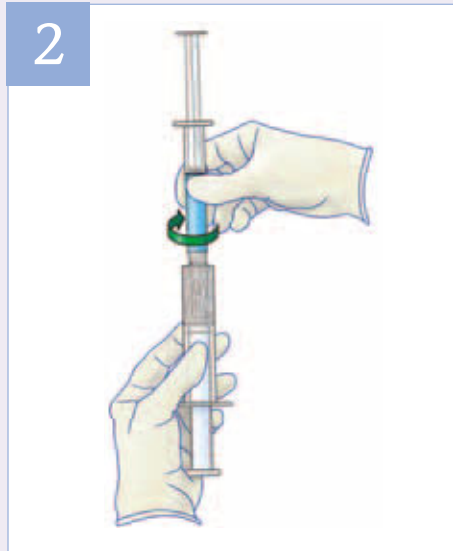


2 Screw and connect the two syringes together.

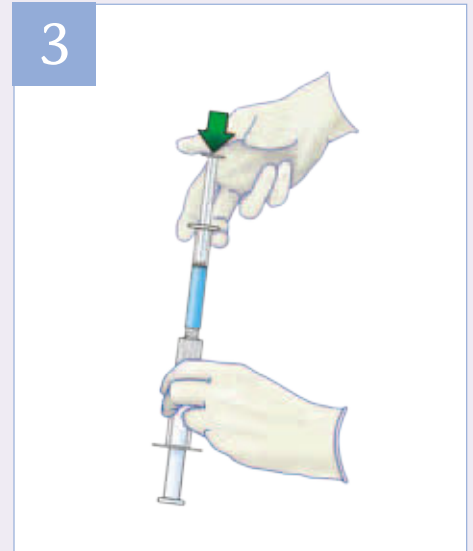

3 With the solution vertically on top, press the small syringe plunger so that all the solution wets the fiber in the large syringe.



4 Mix the foam between the two syringes 10-15 times. **CRITICAL: During each pass, all the foam should move from one syringe to the other.** Finish mixing the foam in the smaller syringe.

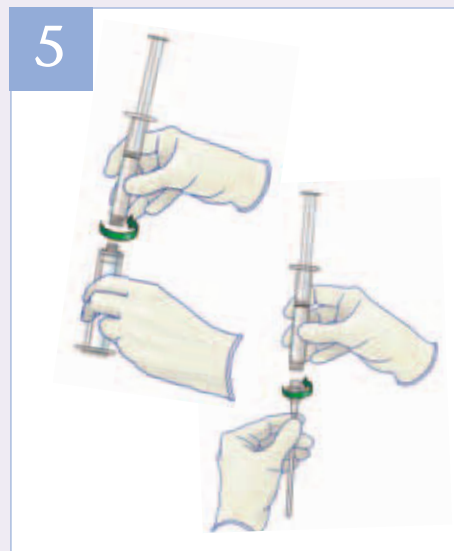

5 Unscrew and disconnect the two syringes from each other. Screw and connect the delivery cannula to the small syringe.



International

ArthroCare ENT
Skeppsbron 2, 111 30 Stockholm, Sweden
Tel: +46 8 546 172 00 Fax: +46 8 546 172 39
ent@arthrocare.se

Australia

ArthroCare (Australasia Pty Ltd)
PO Box 2186, Normanhurst, NSW 2076, Australia
Tel: +61 (0)2 9980 1432 Fax: +61 (0)2 9875 5168

Not for distribution in the United States.

Austria

ArthroCare ENT
Kaufmannstrasse 16, 6020 Innsbruck, Austria
Tel: +43 512 361 538 Fax: +43 512 361538 – 60
info@atlantech.at

France

ArthroCare ENT
9, rue Noirot, 70000 Vesoul, France
Tel: +33 3 84 76 81 38 Fax: +33 3 84 76 74 63
arthrocarefrance@wanadoo.fr

Germany

ArthroCare ENT
Berghauser Str. 62, 42859 Remscheid, Germany
Tel: +49 2191 93342 0 Fax: +49 2191 93342 49
info@arthrocare.de

Scandinavia

ArthroCare Scandinavia ApS
Science Center Skejby, Brendstrupgårdsvej 21
DK-8200 Århus N., Denmark
Tel: +45 7023 2850 Fax: +45 7023 2851
info@arthrocare.dk

UK

ArthroCare UK Ltd
St James Business Park, 9, Grimbold Crag Court
Knaresborough, North Yorkshire HG5 8QB
United Kingdom
Tel: +44 (0) 1423 888806 Fax: +44 (0) 1423 888816
info@arthrocareuk.com