



International gynecology market leader



www.gyneas-international.com
export@gyneas.com
+33 1 42 03 96 77

No. 1 on the French gynecology market



CREATION

Expert in gynecological products since 1999



INNOVATION

Search for new materials (lighter, sharper...)



DEVELOPPEMENT

A range of rich products for liberals and hospitals



CERTIFICATION

Compliance with regulatory standards (certified medical devices)

With 22 years of activity, Laboratoires Gyneas are the French specialist in the design and manufacture gynecological and obstetrical equipment.

Concerned about the lower **quality** of our **equipment**, we take care of **perfection** down to the **smallest details**. This is why we collaborated with the best specialists (midwives, gynecologists, urologists) to create **practical instruments** adapted to their mode of use For the comfort of practitioners and the **well-being of patients**.

OUR CREATIONS



Pessary



Speculum 6 sizes




IUD removal/
insertion set

- More than **250 references**
- **3 to 5 year** warranty
- **Sterile or non-sterile**
- **Stainless steel or titanium**



Obstetrical
vacuum cup

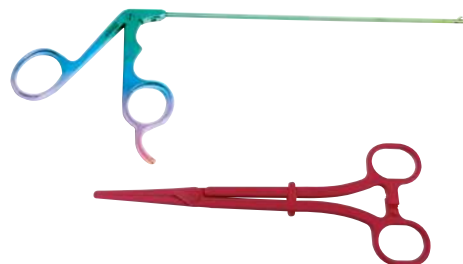

Reusable instrumentation
and single use

GYNEAS

Functional exploration	4 to 6
Sampling devices	7 to 10
Hysterometers.....	11
Procedure kits	12 to 14
Single use instruments.....	15 to 16
Reusable instrumentation	17 to 25
Obstetrics.....	26 to 33
Pessary	34 to 37



SPECULUM

SPECULUM CUSCO

Improves the patient's comfort. This speculum is the reference in the matter of single use device. Softer contact thanks to its rounded edge. No metallic sound; silent manipulation instead. Perfect transparency for the practitioner and flexible security screw with quick locking.

For both comfort and safety of the practitioner and hospital staff, 6 sizes available to better adapt to the different patient's morphologies. Sterilization and expiry dates are mentioned on each pouch.



**Designed and
created with
gynecologists**

BENEFITS :

- Thought and created with gynecologists
- 2 types of packaging
- 6 sizes
- Longer valves for better access to the cervix
- Thicker retaining screw for better blocking
- rounded beak
- Reduction of the opening angle of the jaws so as not to injure the patient during removal
- Articulation of the two reinforced valves

TECHNICAL CHARACTERISTICS :

- Ethylene oxide sterilization
- Polystyrene resin composition
- Latex-free
- Individual sterile sachet packaging



Speculum Cusco

Large packaging per 100

XS	Purple	Ø 12 mm	Length 90 mm	Ref. 70934
Virgin	White	Ø 20 mm	Length 110 mm	Ref. 70936
Small	Blue	Ø 24 mm	Length 116 mm	Ref. 47001
Medium	Red	Ø 26 mm	Length 116 mm	Ref. 47002
Large	Green	Ø 30 mm	Length 116 mm	Ref. 47003

XL	Yellow	Ø 35 mm	Length 131 mm	Ref. 70935
----	--------	---------	---------------	------------

Packaging: box of 50 specula.

Small packaging per 25

XS	Purple	Ø 12 mm	Length 90 mm	Ref. 47848
Virgin	White	Ø 20 mm	Length 110 mm	Ref. 47850
Small	Blue	Ø 24 mm	Length 116 mm	Ref. 70001
Medium	Red	Ø 26 mm	Length 116 mm	Ref. 70002
Large	Green	Ø 30 mm	Length 116 mm	Ref. 70003

XL	Yellow	Ø 35 mm	Length 131 mm	Ref. 47849
----	--------	---------	---------------	------------

Packaging: box of 20 specula.



SPECULUM CUSCO MIX

Speculum Cusco Mix

Compartmentalized. Ref. 70937
Packaging: box of 4 Cusco mix Gyneas specula. 4 sizes: blank, small, medium and large. 25 of each size.
Unit packaging: in a peelable bag



SPECULUM COLLIN

BENEFITS :

- Improved accessibility to the cervix thanks to its side opening
- Optimum visualization thanks to its high quality transparent plastic valves
- Easy and quiet handling
- Practical opener
- Perfect blocking of the valves thanks to a quality life
- 3 different sizes
- Removable for prolapse examinations



Speculum Collin

A Small/Medium Ø 27 mm Length 105 mm Ref. 47214
B Medium/Large Ø 33 mm Length 115 mm Ref. 47215
C Virgin Ø 15 mm Length 95 mm Ref. 46084

Packaging: box of 100 specula (A-B), box of 10 specula (C).
Packaging: Individual sterile pouch.



COMPOSITION :

- Polystyrene resin
- Latex-free
- PVC-free
- Sterilization: ethylene oxide

APPLICATION :

- Gynecological examinations
- Vaginal swabs, smears
- Hysterosonographies and hysterosalpingographies
- Insertion of IUD
- Colposcopy and biopsies

2 PE FINGERSTALL “LEGUEU TYPE”



Non sterile 2 PE Fingerstall

2 PE fingerstall for gynecological exploration, in opaque polyethylene.

Ref. 71029

Packaging: pouch of 100 units



Classe I

Socially clean 2 PE Fingerstall

Ref. 71030

Packaging: pouch of 100 units



Classe I

LUBRICATING JELLY AQUEOUS



COMPOSITION :

- Composition: propylene glycol USP, glycerin USP, Hydroxyethylcellulose, sodium benzoate, polysorbate 20, disodium, EDTA, purified water

BENEFITS :

- Water-based formula
- Respectful of the mucosa
- Does not run, does not stick
- Does not grease or stain
- 3 types of packaging
- Soluble in water

Aqueous lubricating jelly

Designed to lubricate medical devices and to facilitate the introduction of gloves, finger cots or speculum during gynecological examination. The pump bottle allows you to obtain the right dose of jelly for the examination, to prevent the jelly from drying out.

250ml pump bottle.

Ref. 71436

Packaging: box of 50 60 (A), box of 12 (B), box of 100 (C).



Classe I

K-Y JELLY

K-Y sterile lubricating

K-Y jelly is a sterile, non-greasy, transparent, water-soluble and non-irritating jelly. It is fragrance-free, alcohol-free and hypoallergenic.

The K-Y jelly has a pH close to that of the natural environment (5.5). Clinically tested. K-Y jelly can be applied to mucous membranes. It is safe to use with condoms and instruments: probes, cannulas, endoscopes. This jelly is particularly suitable for electrotherapy, cystoscopic procedures and for digital examinations, as well as to facilitate sexual intercourse in case of vaginal dryness.

A Tube 82 g

Ref. 41680

B Set of 8 mini tubes 5 g

Ref. 41679

Packaging: individually.



Classe Is

0123



CERVICAL BRUSH

For the collection and improvement of the endovaginal smear



GYNEAS
Functional exploration

COMPOSITION :

Sterile

- Polypropylene upper.
- Brush: polyamide and stainless steel, latex-free and PVC-free.
- Sterilization: ethylene oxide
- In individual sterile packaging.

Non sterile

- Polypropylene upper.
- Brush: polyamide and stainless steel, latex-free and PVC-free.
- In individual sterile packaging.

BENEFITS :

- 2 brush diameters: 3,5mm and 5,5mm
- With and without ball for the 5,5 mm diameter
- 1 Sterile and 1 non-Sterile range

Cervical brushes

Brush for cytological swabs of the endocervix.
The brush is a swab brush mounted on a polypropylene handle.

Sterile : **A** Ø 3,5 mm. Ref. 39369 // Ø 5,5 mm without ball. Ref. 39368
Packaging: box of 50 units. Individual sterile sachet packaging.

Non sterile : **B** Ø 3,5 mm without ball. Ref. 71559 // Ø 5,5 mm without ball. Ref. 71160 // **C** Ø 5,5 mm with ball. Ref. 71161
Packaging: box of 50 units.



Classe I



Classe Is

ENDOEXOCOL BRUSH

1 gesture, 1 smear, 1 blade = 1 diagnosis!



Endoexocol brush

Allows collection of the endocervix and exocervix in a single rotary motion. Smear in a thin layer or by spreading. Improves smear quality.

Ref. 71162
Packaging: bag of 50.



Classe I

GYNEABIOPS ENDOBIOPS

STANDARD / MINI / PLUS

SAMPLING ENDOMETRIAL BIOPSY

INDICATIONS

Gynebiops enables to diagnose endometrical cancer, hyperplasia, polyps, luteal insufficiency.

GYNEBIOPS STANDARD allows to perform histologic sampling of the endometrium.

GYNEBIOPS MINI allows to perform histologic sampling of the endometrium for patients showing a narrow cervix or a narrow cervical canal and is recommended for post menopausal patients.

GYNEBIOPS CURVE allows to perform histologic sampling on a lateral side of the endometrium and on patients showing a uterus with non-conventional form.

DIRECTIONS FOR USE

- Check the whole packaging and expiring date, which guarantees the sterility and the quality of the product.
- Open the bag, peeling the paper envelop at the point indicated with two arrows.
- Disinfect carefully the cervix.
- With the thin Pozzi forceps, pinch the edge of the cervix.
- Push Gynebiops softly to the desired depth (make use of the graduation).
- Maintain Gynebiops in position, pull the piston back to the stop.
- Sweep across the uterine parts on which one wishes to perform the sampling
- Withdraw Gynebiops, cut off the end of the sheath, keep over a bowl containing adequate preservative liquid and thanks to the bulbous, push the piston towards the distal end to expel the specimen. Send the bowl to the laboratory for analysis.

Gynebiops Endobiops Standard CH9


For microaspiration of endometrial cells.

A Standard Ø 3mm Ref. 46100

Box of 25 units sterile



Classe Is

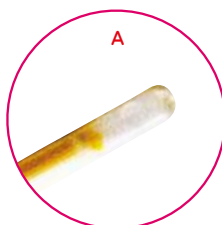

Gynebiops Endobiops Mini CH6


For microaspiration of endometrial cells by patients having a narrow cervix / cervical duct, or atrophied cervix.

B Mini Ø 2mm Ref. 46102

Box of 25 units sterile



Classe Is

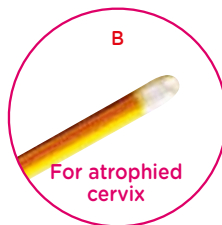

Gynebiops Endobiops Plus CH9

Helps improve the quality of direct debit and/or to make two separate samples.

C Plus Ø 3mm Ref. 47693

Box of 25 units sterile



Classe Is



FORCEPS




BIOPSY CERVICAL



Gyn & Push

Your successful
biopsies with a boost!

GYNEAS
Sampling

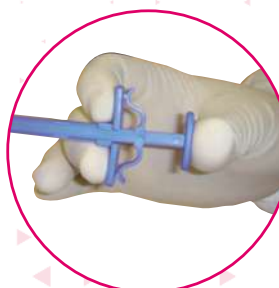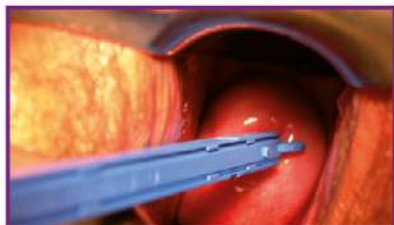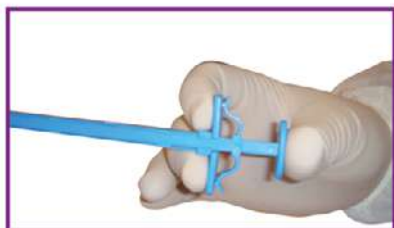

Cervical biopsy forceps

Before use Refer to the instructions for use.


Precision and efficiency by adopting a single gesture:
a simple thumb pressure. Each forceps, with its plastic
needle, is individually wrapped.


Ref. 76021

Packaging: box of 25 units



Classe IIa

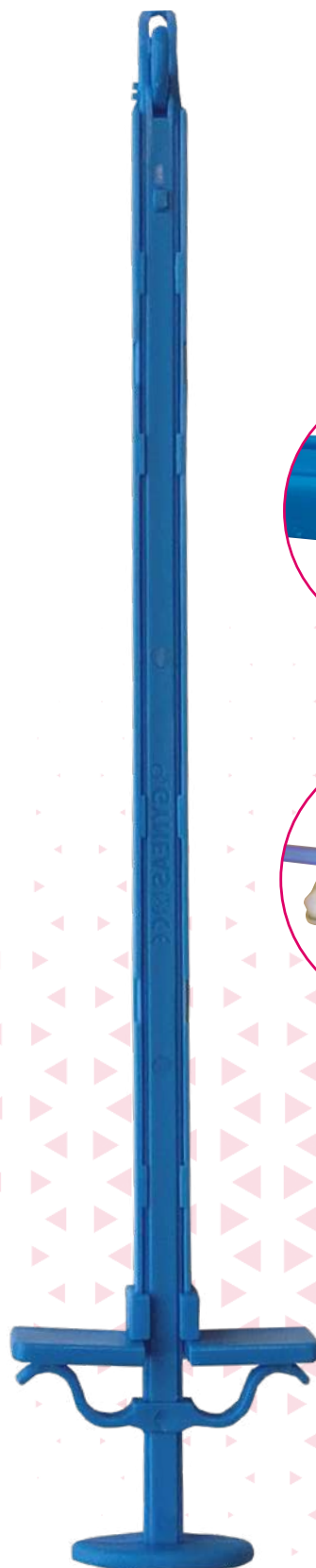

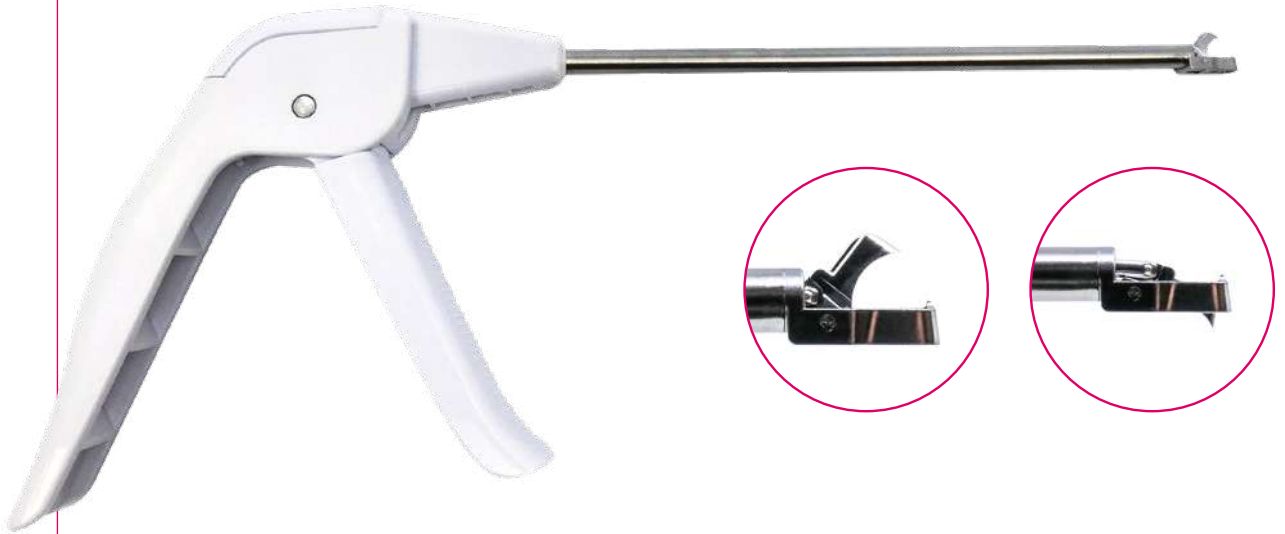

Cervical biopsy forceps

This biopsy forceps is a Gyneas innovation and revolutionizes the field of single use thanks to its gripping quality and excellent cut.

Ref. 51192

Packaging: box of 10 units



TECHNICAL CHARACTERISTICS :

- Material: stainless steel
- Biopsy forceps
- Rigid blister

BENEFITS :

- Same quality as your reusable instruments
- Excellent fit
- Allows direct debit sample accuracy
- Ergonomic handle
- Easy opening
- Rigid blister to protect the clamp



GYNEAS HYSTEROMETER

DUAL EVOLUTION



GYNEAS
Sampling

Hysteroscope

Ref. 46315
Packaging: box of 25.



TECHNICAL CHARACTERISTICS :

- Very flexible extremities with shape memory
- Each end is protected by a transparent tube
- Unit packaging in a peelable bag

BENEFITS :

- 1 single hysteroscope for all IUD / IUS insertions
- 1 side CH10 to CH14
- 1 side CH8 to CH12

HYSTEROMETER

Hysteroscope CH10

Soft distal part on a rigid handle.
A Ø 3,3mm Ref. 39246
Box of 25 units sterile



Hysteroscope CH12

Soft distal part on a rigid handle.
B Ø 4,3mm Ref. 39247
Box of 25 units sterile






Hysteroscope CH14

With a nozzle of progressive dilatation, from CH10 to CH14 over 25 mm. Indicated for the fitting in of the progestogen IUD (intra-uterine contraceptive device). Dilates the endocervix and measures the depth of the uterine cavity.
C Ø 4,67mm Ref. 39248
Box of 25 units sterile



GYNEAS STERILE COOPER IUDS

TCU 380 + AND 375 CU

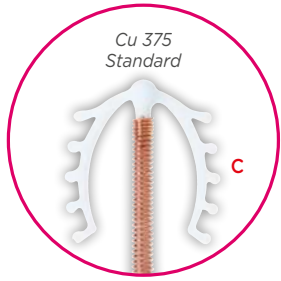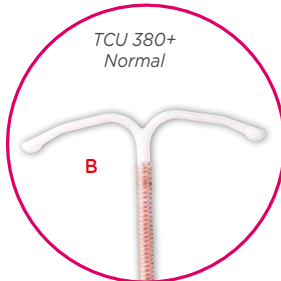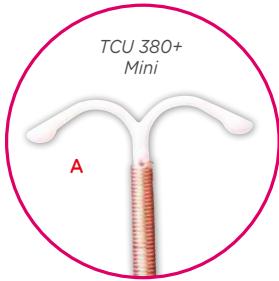

Copper IUD

- A** TCU 380+ Mini - Ref. 46106
- B** TCU 380+ Normal - Ref. 46105
- C** CU 375 Standard - Ref. 46107

Packaging: individually.



TECHNICAL CHARACTERISTICS :

A-B TCU 380+

- Mixture of low density polyethylene and barium sulphate, wrapped in copper wire.
- The T has a single filament polyamide thread (suture) for easy removal.
- Packaged with an inserter and a pusher in a peelable bag. The inserter is fitted with a sliding collar to help gauge how deep it should be inserted into the uterine cavity through the cervical canal

C CU 375

- Polyethylene, stem extended by two flexible branches fitted with lugs. The stem is wrapped in copper wire. The flexible side arms ensure that the Cu 375 is held as high as possible deep in the uterus without stretching the uterine cavity in any way. A single filament nylon thread is attached to the rod.

The TCU 380+ (Mini or Normal) and CU 375 IUDs can be left in place for a maximum of 60 months (5 years).

IUD HOOK



TECHNICAL CHARACTERISTICS :

- Length 26cm.
- Unit packaging in a peelable bag.

BENEFITS :

- Benefits: Specially designed to be inserted into the uterine cavity and hook the thread of the IUD.

IUD hook

Ref. 47300
Packaging: box of 20.



LONG SCISSORS WIRE CUTTERS



TECHNICAL CHARACTERISTICS :

- Handle is made of rigid plastic; metal cutting part. Length: 22cm
- Sterilization: with ethylene oxide

BENEFITS :

- Specially designed to cut the threads of the IUD after it has been inserted.

Long wire cutter scissors

Ref. 47299
Packaging: box of 25 scissors



IUD KIT

IUD INCLUDED (COPPER)

COMPOSITION :

- 1 copper IUD (optional)
- 1 Evolution IUD insertion set containing:
 - 1 Pozzi forceps - 1 long forceps
 - 1 thread cutting scissors - 1 double hystrometer
 - 1 Cusco Medium speculum 26 mm Gyneas.

BENEFITS :

- Complete package
- 3 sizes available

IUD Kit - Complete Pack

A Complete range that corresponds to all women

A Kit TCU 380+ Mini Ref. 47539

B Kit TCU 380+ Normal Ref. 47079

C Kit CU 375 Standard Ref. 47542

Packaging: individually.



TCU 380 et Cu 375
MD CE
Classe III 2460

Set de pose DIU
MD CE
Classe IIa 0476

Speculum
MD CE
Classe Is 0123

Complete insertion set: IUD included at the same price as the IUD alone




GYNEAS
Single-use instruments
& IUD installation set

IUD INSTALLATION SET

EVOLUTION

Evolution with double hystrometer

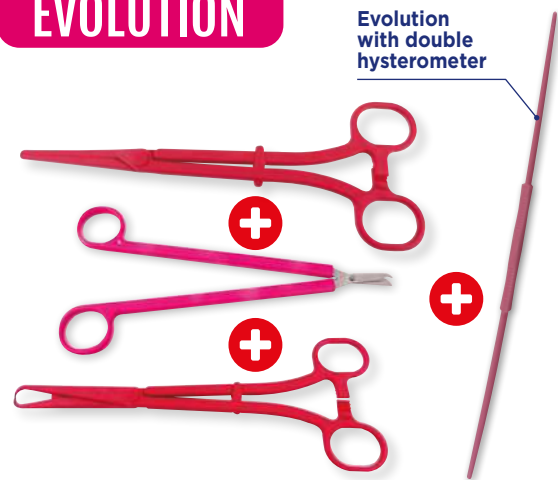

COMPOSITION :

- 1 long pliers - length 25 cm
- 1 Pozzi forceps - length 25 cm
- 1 Evolution double hystrometer - length 30 cm
- 1 thread cutter with notch to block the IUD thread between the 2 metal blades - blade length 22 cm in stainless steel

Evolution IUD insertion set

Ref. 46296


Packaging: individually in box of 10



1282

IUD INSTALLATION SET

Classic without hystrometer

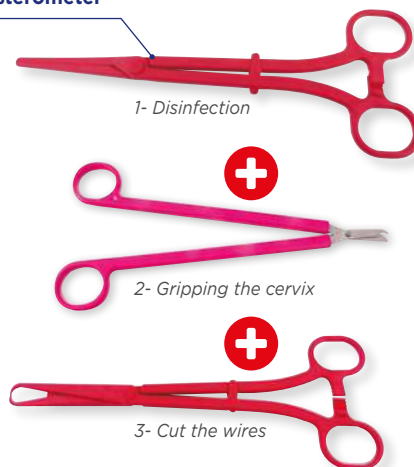

1- Disinfection

2- Gripping the cervix

3- Cut the wires

COMPOSITION :

- 1 long pliers - length 25 cm
- 1 Pozzi forceps - length 25 cm
- 1 thread-cutting scissors with notch to lock the IUD thread between the 2 metal blades - length 22 cm in stainless steel.

IUD installation set

Ref. 46293

Packaging: box of 10.



1282

CONTRACEPTIVE IMPLANT REMOVAL

REMOVAL SET EXTRIMPLANT®

COMPOSITION :

- 5 non-woven compresses 7.5 x 7.5 cm 30 g
- 1 curved Halstead Mosquito forceps 12 cm
- 1 retractable syringe with 25G x 5.8 mm needle
- 1 No. 11 scalpel
- 1 strip of 3 adhesive skin sutures
- 1 compression bandage 5 x 4 m
- 1 Green perforated drape with adhesive 50 x 60 cm
- 1 Extrimplant forceps

Extrimplant Gyneas Removal Set


Designer Dr. Henri Levy (Paris).

Ref. 46310

Packaging: box of 10 sets



An innovation
GYNEAS

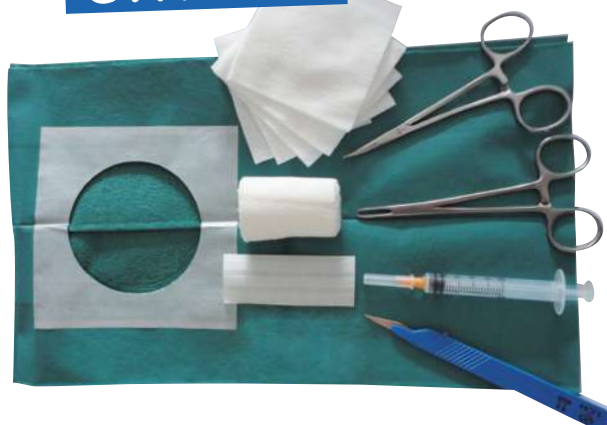

REMOVAL SET REGULAR

COMPOSITION :

- 1 adhésive fenestrated drape 50 x 60 cm
- 1 No. 11 scalpel
- 2 non woven swab 7.5 x 7.5 cm
- 1 strip of 3 adhesive skin sutures
- 1 compression bandage
- 1 syringe 2ml
- 1 hypodermic needle
- 1 curved Halstead forceps (mosquito)

Gyneas implant removal set

Ref. 46308

Packaging: box of 10 sets



INSTALLATION SET IMPLANT

COMPOSITION :

- 1 perforated adhesive drape
- 1 syringe 2ml
- 1 hypodermic needle 16 x 0.5 cm
- 1 hypodermic needle 25 x 0.8 cm
- 2 non-woven compresses 7.5 x 7.5 cm
- 1 sticking plaster 3M 5 x 7.2 cm

Gyneas implant placement set

Ref. 46312

Packaging: box of 10 sets



FORCEPS

POZZI

Resistant, fine and benevolent

TECHNICAL CHARACTERISTICS :

- Reduction of the thickness of the forceps so that it takes up less space in the speculum
- Thinner and stronger jaw tips with added fiberglass in the material
- An additional notch in the clamp closing system to give the practitioner more degrees of freedom
- Improved handling and ergonomics



APPLICATION :

- Used during certain procedures such as cerclage, biopsies, conization
- Traction of the cervix to bring its axis closer to that of the cervix


Pozzi forceps

Length 25 cm.

Ref. 44778. Packaging: box of 25 units

Ref. 46292. Packaging: box of 50 units

Unit packaging in a peelable bag.

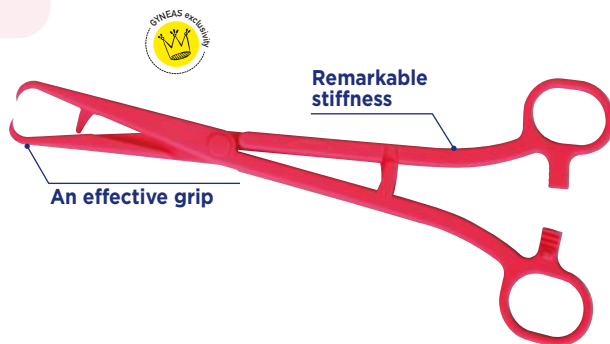

CHERON

TECHNICAL CHARACTERISTICS :


- In polystyrene resin
- Sterilization with ethylene oxide
- Angled forceps for tampon or compress holder for disinfection of the cervix

Cheron forceps

Length : 25 cm. Ref. 47291

Packaging: box of 25 units

Unit packaging in a peelable bag.

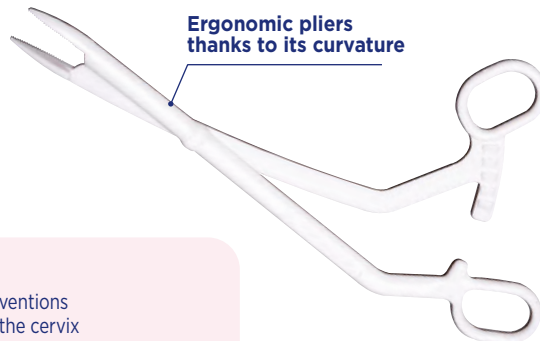

APPLICATION :

- Gynecological interventions
- Performing care at the cervix
- It is an angled forceps carrying a tampon or a compress allowing the disinfection of the cervix, before examination, and can be used more generally for the care of wounds.

LONG BENGOLEA EVOLUTION

TECHNICAL CHARACTERISTICS :

- In polystyrene resin
- Thinner than a classic long forceps, it can be introduced into the endocervix.
- Modified jaws to improve grip of IUD threads.
- Sterilization with ethylene oxide.

Evolution long bengolea pliers

Length : 25 cm. Ref. 46301

Packaging: box of 25 units

Unit packaging in a peelable bag.



APPLICATION :

- Intended to disinfect the cervix before any intervention, such as for the insertion of an IUD.
- Some practitioners also use it to pull on the IUD string during removal. Wedge the wires between the jaws, make a complete rotation of the wires on the jaws and pull.



Very thin jaws

GYNECOLOGICAL FORCEPS AND HOOK

DISPOSABLE

METZENBAUM THREAD CUTTING SCISSORS

TECHNICAL CHARACTERISTICS :

- Curves and round blades.
- Length: 180mm

Metzenbaum thread cutting scissors

Ref. 76077

Packaging: box of 10 pincers



APPLICATION :

- The blade has a notch to block and cut the thread of the sterile



CHERON FORCEPS

TECHNICAL CHARACTERISTICS :

- High quality steel
- Length: 240mm
- Serrated jaw

Cheron forceps

Ref. 46495

Packaging: box of 10 forceps



APPLICATION :

- Locking system for gripping compresses and dressings during disinfection of the cervix during gynecological examinations.



BENGOLEA FORCEPS

TECHNICAL CHARACTERISTICS :

- Serrated jaws
- Without claws
- Steel
- Length: 240mm

Bengolea forceps

Straight.

Fine model. Ref. 47304

Packaging: box of 10 forceps



APPLICATION :

- Haemostatic forceps generally used to clamp vessels. Its serrated jaws allow you to grasp and hold effectively



POZZI FORCEPS

TECHNICAL CHARACTERISTICS :

- Steel
- Length 240 mm with claw
- 2 types of jaws: fine or ultra fine

Pozzi forceps

Straight, without claw. Length : 240 mm.

A Thin model. Ref. 47302


B Ultra thin model. Ref. 46494

Packaging: box of 10 forceps



APPLICATION :

- Apply pressure to the cervix to grasp the cervix and then fix it during certain procedures.
- Placement of an IUD
- Equipped with an effective claw, access to the uterine cavity can be made without damaging the uterus.
- Helps the gynecologist in the detection of endocrine disease, uterine or ovarian cancer.

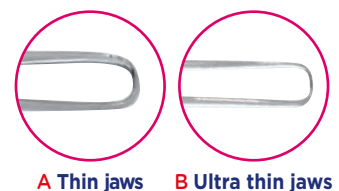


A Thin jaws **B** Ultra thin jaws



THE OF TITANIUM INSTRUMENTS

-  **Lighter**
-  **More robust**
-  **More precise, better cutting**
-  **Require less sharpening**
-  **Do not scratch when cleaning**
-  **Autoclavable like conventional instrumentation**
-  **Very aesthetic, with four-color reflections depending on the light**

IDENTIFY GYNEAS TITANIUM PRODUCTS USING THE FOLLOWING PICTOGRAMS

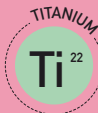

TITANIUM

Reusable instrumentation

Novak cannula.....	20
Scissors (Mayo type / Allaines Metzenbaum / Metzenbaum wire cutters)	20
IUD hook.....	20
Kevorkian curette.....	20
Dilator.....	24
Vaginal retractor.....	18
Forceps	25
Hysterometer.....	23
Pince.....	21 to 22
Spatules	25
Speculum	18 to 19
Vaginal valves.....	23



SPECULUM REVU



STAINLESS
STEEL



TITANIUM

COLLIN SPECULUM



One-Piece Collin Speculum

Speculum Collin one-piece stainless steel


Useful length 16 x 90 mm	Ref. 70990	//	Useful length 26 x 90 mm	Ref. 70992	//	Useful length 30 x 90 mm	Ref. 70994
Useful length 32 x 105 mm	Ref. 70999	//	Useful length 35 x 105 mm	Ref. 71003	//	Useful length 38 x 105 mm	Ref. 71005

Packaging: individually



CUSCO SPECULUM FOLDING

A complete range that suits the morphology of all your patients

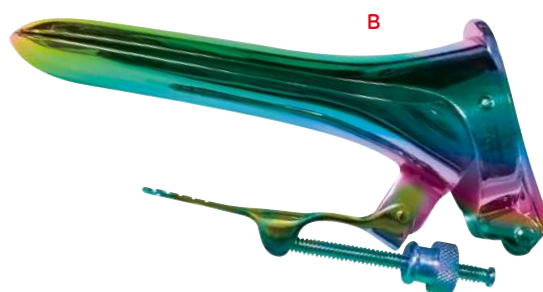

Folding Cusco Speculum

A Stainless steel folding Cusco speculum

16 x 100 mm	Ref. 71007	//	20 x 100 mm	Ref. 71009	//	25 x 100 mm	Ref. 71011	//	28 x 100 mm	Ref. 71015
30 x 100 mm	Ref. 71019	//	32 x 100 mm	Ref. 71227	//	36 x 100 mm	Ref. 71231	//	25 x 115 mm	Ref. 71013
28 x 115 mm	Ref. 71017	//	30 x 115 mm	Ref. 71225	//	32 x 120 mm	Ref. 71230			

B Speculum Cusco folding titanium

16 x 100 mm	Ref. 71008	//	20 x 100 mm	Ref. 71010	//	25 x 100 mm	Ref. 71012	//	28 x 100 mm	Ref. 71016
30 x 100 mm	Ref. 71224	//	32 x 100 mm	Ref. 71229	//	36 x 100 mm	Ref. 71232	//	25 x 115 mm	Ref. 71014
28 x 115 mm	Ref. 71018	//	30 x 115 mm	Ref. 71226	//	32 x 120 mm	Ref. 71228			

Packaging: individually




VAGINAL RETRACTOR LATERAL



APPLICATION :

- For an unobstructed view of the cervix
- Allows easy and smooth removal of the prolapse from the vaginal wall without removing the speculum
- Complementary instrument to the speculum in the event that the opening with the two valves of the speculum is not sufficient
- Can also be used alone to spread the vaginal walls

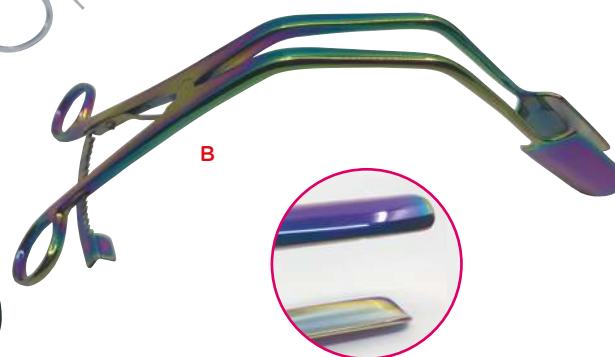

Vaginal retractor lateral

A Stainless steel - Ref. 70988
B Titanium - Ref. 70989


Packaging: individually

SPECULUM REU



Possibility to adjust the height of the upper valve

PEDERSON TITANIUM

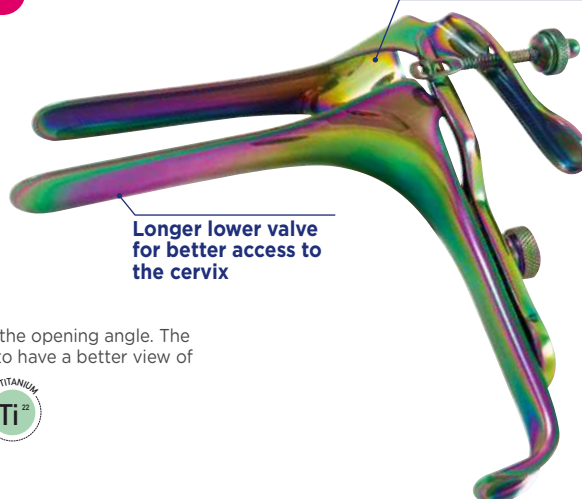

Longer lower valve for better access to the cervix

Titanium Pederson Speculum


Possibility to select the height of the valves in addition to the opening angle. The lower valve is a little longer than the upper valve in order to have a better view of the cervix. 4 different sizes.

75 x 13 mm. Ref. 71238 // 100 x 16 mm. Ref. 71234
100 x 22 mm. Ref. 71235 // 120 x 25 mm. Ref. 71236

Packaging: individually



ENDOCERVICAL KOGAN

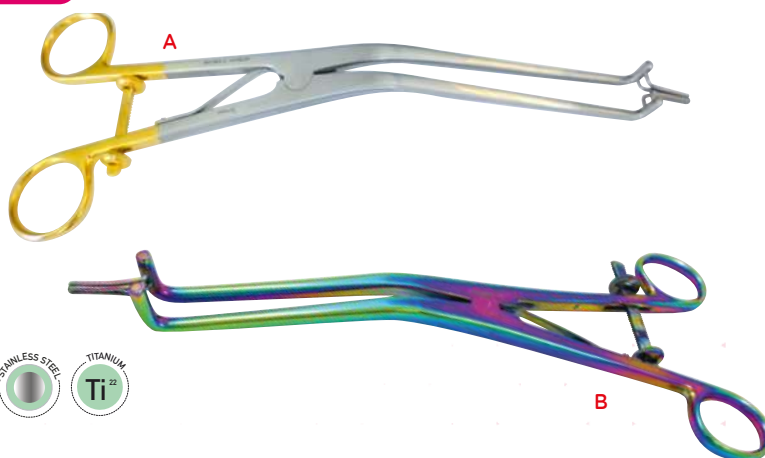

Speculum endocervical Kogan

Width of the valves 3 mm.
Length 240mm. Reusable.

A Stainless steel - With rack. Ref. 71233

B Titanium - With rack. Ref. 71356

Packaging: individually





4 VALVES®



The 4 valve speculum®

An innovation
GYNEAS

An instrument for
the management of
postpartum hemorrhage
(Dr. Jonathan Cohen).

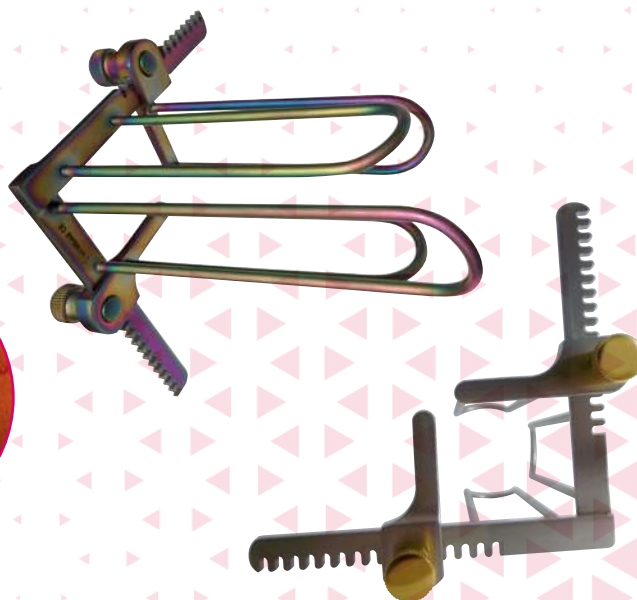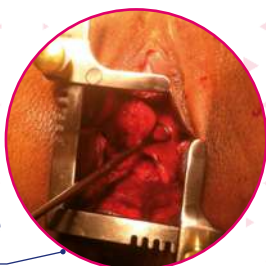

Speculum 4 valves®

In the context of the haemorrhage of deliverance, it is necessary to act quickly and well. Dr. Jonathan COHEN has devised a device allowing the obstetrician to expose and analyze the birth canal without the contribution of outside help. With this speculum, the obstetrician can perform cervical revision or any deep vaginal suture alone.

Small model - Length 3 valves 110 mm - Length 1 valve 130 mm - Length 4 valves 23 mm

Stainless steel. Ref. 71301 // Titanium. Ref. 71302

Large model - Length 3 valves 108,60 mm - Length 1 valve 133,40 mm - Length 4 valves 34,50 mm

Stainless steel. Ref. 71299 // Titanium. Ref. 71300

Packaging: individually

INSTRUMENTATION **REU**



METZENBAUM SCISSORS

WIRE CUTTERS

Metzenbaum thread cutting scissors

For their finesse, these scissors are recommended for cutting the IUD thread after insertion. A wire-cutting notch has been added to one of the blades at its end so that the wires hug the notch and are cut with a single movement.

Length : 230 mm.

A Straight - Stainless steel - Ref. 71324 // Straight - Titanium - Ref. 71330
B Curve - Stainless steel - Ref. 71327 // Curve - Titanium - Ref. 71328

Packaging: individually



ALLAINE TYPE

Allaines Metzenbaum type scissors

Curved model 180 mm Ref. 71484 // 200 mm Ref. 71342 // 230 mm Ref. 71343
 Modèle Straight 140 mm Ref. 71491 // 200 mm Ref. 71341 // 230 mm Ref. 71367

Packaging: individually



TYPE MAYO

Mayo scissors

Complete range of scissors offering different lengths in Straights or Curves version.

Mayo scissors Stainless steel

Straight 140 mm Ref. 71369 // Straight 170 mm Ref. 71370 // Straight 230 mm Ref. 71357
 Curve 140 mm Ref. 71368 // Curve 170 mm Ref. 70985 // Curve 230 mm Ref. 71371

Packaging: individually



CANULE TYPE NOVAK



Enduterine biopsy/ IUD removal

APPLICATIONS :


- Cannula can also be used to hook IUD threads thanks to the small teeth at its end

Canule type Novak

Length : 203 mm.

A Stainless steel : Ø 2 mm Ref. 71470 // Ø 3 mm Ref. 71285 // Ø 4 mm Ref. 71001
B Titanium : Ø 2 mm Ref. 71284 // Ø 3 mm Ref. 71286 // Ø 4 mm Ref. 71318

Packaging: individually

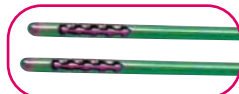

IUD HOOK



IUD hook

Length : 260 mm.

A Stainless steel -Ref. 71283
B Titanium - Ref. 71475

Packaging: individually



APPLICATIONS :

- Device for hooking the threads of the IUD or IUS when they are no longer visible and have gone up into the cervix

KEVORKIAN CURETTE



Kevorkian curette

Length : 300 mm.

Stainless steel thin version, basket 10 x 2 mm Ref. 71462
 Titanium classic version, basket 12 x 3 mm Ref. 71297

Packaging: individually



TECHNICAL CHARACTERISTICS :

- Basket to collect the sample
- Edges curved inwards and slightly blunt

APPLICATIONS :

- Used in colposcopy to take samples from the endocervix

FORCEPS REVU



STAINLESS
STEEL

TITANIUM

CICERON

An innovation
GYNEAS

For easy removal of the IUD, even
when the strings are invisible

BENEFITS :

- Simplify the removal of the IUD, in particular when the strings are no longer visible or have gone up into the cervix. The length and fineness of the forceps are ideal for introducing it into the cervix and grasping the IUD threads


Pince Cicéron

Length : 210 mm. Total length: 275 mm.

A Stainless steel - Ref. 71287

B Titanium - Ref. 71288

Packaging: individually

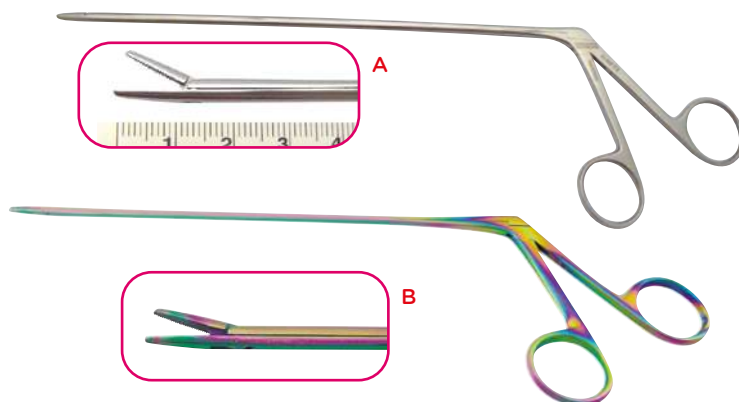

TERRHUN

BENEFITS :

- This clamp can be completely disassembled for rigorous hygiene.

APPLICATIONS :

- To crochet the IUD when the threads are absent


Terrhun Claw

Length : 270 mm.

Stainless steel - Ref. 71313

Titanium - Ref. 71289

Packaging: individually

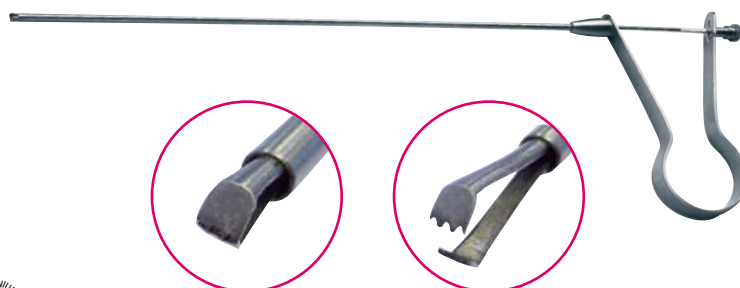

BIOPSY METAL




MICRO TISCHLER

An innovation
GYNEAS



Micro

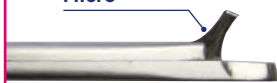

Tischler micro metal biopsy forceps

Jaw dimensions: 3,5 x 1,2 mm.

Clamp length 180 mm.


Stainless steel.

Ref. 71310

Titanium. Ref. 71311

Packaging: individually

MINI TISCHLER

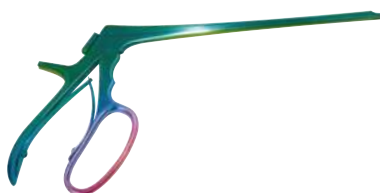

Mini



Biopsy forceps metal mini Tischler

Jaw dimensions: 2,3 x 4,2 mm.

Clamp length 200 mm ou 250 mm.

Stainless steel : 200 mm Ref. 70996

250 mm Ref. 70997

Titanium : 200 mm Ref. 71307


250 mm Ref. 70998

Packaging: individually

CLASSIC TISCHLER



Classic

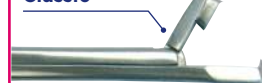

Metal biopsy forceps Tischler classic

Jaw dimensions: 3 x 7 mm.

Clamp length : 200 mm.

Stainless steel.

Ref. 71308

Titanium. Ref. 71309

Packaging: individually

FORCEPS **REU**



STAINLESS
STEEL

TITANIUM

BENGOLEA FORCEPS WITHOUT CLAW

Bengolea forceps without claw

Length 240 mm.

Stainless steel : Curve - Ref. 71305 // Straight - Ref. 71303

Titanium : Curve - Ref. 71306 // Straight - Ref. 71304

Packaging: individually



ALLIS

Allis forceps

Stainless steel hemostatic forceps,
surgical grade.

Straight, length 25 cm - Ref. 71485

Curve, length 23 cm - Ref. 71312

Packaging: individually



PALMER **POZZI**

Ultra thin

B

Pozzi Palmer

Uterine Traction length 240 mm.

A Stainless steel : Classic - Ref. 70964 // Very thin - Ref. 71314

B Titanium : Classic - Ref. 70965 // Very thin - Ref. 71291

Packaging: individually



A

PALMER **MUSEUX**

Palmer museux forceps


2 jaws to properly grasp and hold the cervix.
Length 230mm. 5 mm jaws.

Ref. 70963

Packaging: individually



Uterine traction
forceps

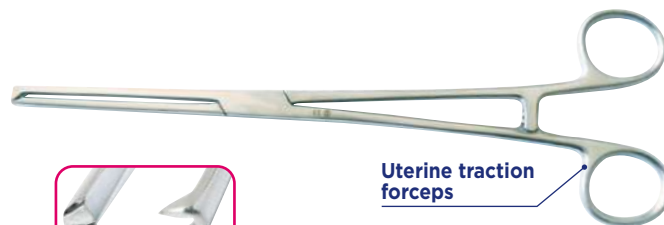

PALMER **POLYP**

Palmer Polyp Forceps

Hollow and fine ends to block the polyp and detach
it by practicing scalpel with forceps.

Length 220 mm.

Stainless steel - Ref. 71355 // Titanium - Ref. 71290

Packaging: individually



DRESSING

Dressing forceps

A Stainless steel : Straight 240 mm - Ref. 71486

Straight 300 mm - Ref. 71488

Curve 240 mm - Ref. 71487

B Titanium : Straight 240 mm - Ref. 71464

Packaging: individually



A



B



HYSTEROMETER REU



STAINLESS STEEL

BENEFITS :

- Estimate the fundal height before, for example, the insertion of an IUD.
- Gentle dilation of the cervix



Hysterometer

A Malleable Sims 320 mm - Ref. 71298

B Valleix malléable 290 mm - Ref. 71315

Packaging: individually



VAGINAL VALVES REU



DOYEN



Doyen vaginal valve

Length : 240 mm. Stainless steel.

55 x 35 mm - Ref. 71242 // 85 x 35 mm - Ref. 71245 // 115 x 35 mm - Ref. 71240

55 x 45 mm - Ref. 71243 // 85 x 45 mm - Ref. 71246 // 115 x 45 mm - Ref. 71241

55 x 60 mm - Ref. 71244 // 85 x 60 mm - Ref. 71247

Packaging: individually



DELEE



Delee vaginal Valve

Length 23,5 cm.

Stainless steel - Ref. 71239

Packaging: individually



DILATOR REU

HEGAR SINGLE CANDLE



STAINLESS
STEEL



TECHNICAL CHARACTERISTICS :

- Stainless steel
- Length : 20 cm
- Diameter: 1 to 30 mm
- 30 sizes available
- Each size is 1mm extra in diameter

APPLICATION :

- Extemporaneous dilation of the cervix and/or measurement of the diameter of the opening of the cervix (possibility of passing a Hegar n°8 candle)
- Vaginal dilator, for example as part of a treatment for vaginismus

Hegar Single Candle

Medical instrument in reusable stainless steel, for the extemporaneous dilation of the cervix and/or the measurement of the diameter of the opening of the cervix. Also used as a vaginal dilator, for example in the treatment of vaginismus.

Ø N°1	Ref. 70952
Ø N°2	Ref. 70966
Ø N°3	Ref. 70977
Ø N°4	Ref. 70979
Ø N°5	Ref. 70980
Ø N°6	Ref. 70981
Ø N°7	Ref. 70982
Ø N°8	Ref. 70983
Ø N°9	Ref. 70984
Ø N°10	Ref. 70953
Ø N°11	Ref. 70954
Ø N°12	Ref. 70955
Ø N°13	Ref. 70956
Ø N°14	Ref. 70957
Ø N°15	Ref. 70958

Ø N°16	Ref. 70959
Ø N°17	Ref. 70960
Ø N°18	Ref. 70961
Ø N°19	Ref. 70962
Ø N°20	Ref. 70967
Ø N°21	Ref. 70968
Ø N°22	Ref. 70969
Ø N°23	Ref. 70970
Ø N°24	Ref. 70971
Ø N°25	Ref. 70972
Ø N°26	Ref. 70973
Ø N°27	Ref. 70974
Ø N°28	Ref. 70975
Ø N°29	Ref. 70976
Ø N°30	Ref. 70978

Packaging: individually



FORCEPS & SPATULAS **REU**



STAINLESS
STEEL

OBSTETRIC FORCEPS

DEMLIN SUZOR

Obstetric forceps Demelin Suzor

Stainless steel. 320 mm.
Ref. 47534

Packaging: individually



OBSTETRIC FORCEPS

TARNIER WITH TRACTOR

Obstetric forceps Tarnier with tractor

Stainless steel. 400 mm.
Ref. 46821

Packaging: individually



OBSTETRIC FORCEPS

PAJOT 360 MM

Obstetric forceps Pajot 360mm

Stainless steel. Ref. 47535

Packaging: individually



APPLICATION :

- Used mainly in the lower part of the excavation but also for instrumental extractions during caesareans



DELIVERY SPATULA

THIERRY

TECHNICAL CHARACTERISTICS :

- Two independent levers, without perforation, without Clamping mediums
- 1 handle and 1 spoon form the actual spatula.

APPLICATIONS :

- Spread the maternal tissues in front of the fetal head

Thierry spatulas Stainless steel 360 mm

Stainless steel. Ref. 46999

Packaging: the pair



OBSTETRICS & PESSARY

OBSTETRICS :

Clamp de Bahr	31
Curette	33
Sleeve for abortion set.....	32
Fetal monitor	27
Amniotic Perforator	31
Clamp Cutter Pliers	31
Cord care set	31
Abortion set	32
Ultrasound Gel	28
Probe Covers	29
Icup ² Obstetric Vacuum Cup	30

PESSARY :

Cubic.....	35
Dish.....	36
Donut.....	36
Gyn&Cube.....	34
Ring	36



PROFESSIONAL FETAL PULSE DETECTOR

For your consultations
in the office or at home



BENEFITS :

- Nice shape, portable, easy to use
- Rounded probe to ensure easy use and comfort for pregnant women
- Heart rate, bar graph and pulse wave values are displayed on the color screen
- The value is displayed in Red when the heart rate is outside the normal measurement range

TECHNICAL CHARACTERISTICS :

- Color screen display, liquid crystal LCD size 45 x 25 mm. 160 x 128 pixel resolution
- Battery charge indicator
- Digital heart rate data
- Nominal probe frequency
- Work mode
- Heart rate measurement range 50 to 240 bpm (beats/minute) accuracy +/- 2 bpm
- 1 bpm resolution
- Fetal labor frequency 2 MHz (+/- 10%), vascular 5 or 8 MHz (+/-10%)
- Isata fetal output intensity < 5 mW/cm2, Isata vascular < 10 mW/cm2
- Power supply 2 x 1.5V LR6 AA batteries (included)
- Manual or automatic shutdown after 1 min without signal
- Type B electric shock protection
- IPX4 splash
- Operating temperature +5 to +40°C
- Dimensions (L x W x H in mm): 135 x 92 x 29.245 g (with batteries)
- Complies with European Directive 93/42 EEC 2007/47/EC relating to medical devices

Professional Fetal Pulse Detector

Portable obstetrical device that can be used in hospitals, clinics or private practices.

Doppler with color screen + 2MHz probe - Ref. **44927**

Battery Varta Longlife Maxpower AA/LR6 X4 - Ref. **51171**

A Vascular probe 5 MHz- Ref. **37311** // **B** Vascular probe 8 MHz - Ref. **37312**



ULTRASOUND GEL

FLACON



Supragel flacon

Water-based non-sterile gel used as contact media in non-invasive, low risk medical procedures (diagnostic or therapeutic) where a viscous gel is required.

Ref. 71429 Packaging : Box of 40 bottles (250ml)

Ref. 48545 Packaging : Individually bottle (250ml)



CUBITAINER

A



B



Supragel cubitainer

Water-based non-sterile gel used as contact media in non-invasive, low risk medical procedures (diagnostic or therapeutic) where a viscous gel is required.

A Ref. 71428

Packaging : 2 flexible containers 5L + 2 empty bottles + 2 nozzles + 1 pump

B Ref. 48546 Packaging : 5L container



PROBE COVERS

NON STERILE



B



Non-sterile probe covers

Non-sterile latex. Pure latex probe covers.

Dimensions (without extension):

length 195mm, diameter 33mm.

Not sterile, not lubricated or lubricated.

A Not lubricated Ref. 47613 / **B** Lubricated - Ref. 46111

Packaging: Box of 144 units.



STERILE



Sterile probe covers

Sterile latex probe covers

Dimensions: length 200mm.

Diameter 20mm. In individual blister bag. Not lubricated

Ref. 46077

Packaging: Box of 100 units.



NON STERILE

LATEX FREE



Latex-free probe covers

Not sterile. Polyisoprene

Not lubricated. Indicated for people allergic to latex.

Ref. 46113

Packaging: Box of 72 units.



A NEW DESIGN for better functionality

*The iCup2® obstetrical vacuum cup is an instrumental assistance device for the fetus.
For single use, it is sterile and ready to use under the conditions of use.
and for recommended indications.*

Description :

The iCup2® suction cup comprises a cup-shaped vacuum chamber with a 5 cm diameter and 2.2 cm height opening with a pull lake, an ergonomic handle and a connection tube on the vacuum pump.

The outer and inner surfaces of the cup are smooth so as not to traumatize the child or the patient's birth canal. The inside of the cup contains a soft, smooth silicone disc to protect the fetal scalp.

On the dome of the cup there is a notch specially designed for the operator's finger and to allow him to orient himself during insertion and during fetal progression.

The suction inside the cup is circumferential to the internal roof for optimized efficiency and to eliminate any risk of obstruction of the suction system. The traction lake, made up of a 22 cm medical cord, is located exactly in the center of the cup dome with 180° mobility in the frontal plane.

The ergonomic handle is rounded for a good grip. Possibility of reducing the length of the traction lake by turning it on the handle. The flexible suction pipe which starts from the dome of the cup, is 1 m long (to allow complete freedom of movement for the operator), ends in a variable diameter adapter designed to connect to the majority of pumps empty available. The initial part of this hose is covered with a Greene color mark which must remain partially visible during installation, thus guaranteeing application in the pelvic excavation and not above.

Indications :

The main indications for application of the iCup2® obstetrical vacuum cup are:

- Arrest of progression of the fetal head in the excavation.

The arrests of the descent of a cephalic presentation at the level of the Medium strait are very often defects of rotation of a posterior position and the complement of flexion obtained by the suction cup allows rotation in occipito-pubic and vaginal delivery. .

- Assistance with eviction.
- Uterine inertia in addition to oxytocics.
- Any anomalies of the fetal heart rate (FHR) justifying an instrumental extraction.

Remarks : Always consider fetal well-being before making an indication for instrumental extraction. Significant heart rate abnormalities should lead to reconsideration of the indication for fetal extraction based on the risk/benefit calculation.

Traceability ensured by its 4 self-adhesive labels present on each packaging for the patient file.

Sterilization with ethylene oxide.

iCup²
The perfect gesture



The iCup2® trademark is registered.

**Innovation
and French made**

Disposable sterile obstetrical suction cup

Interior and exterior surfaces of the cup are smooth, so as not to traumatize the child or the patient's birth canal.

Notch for user's finger for orientation during fetal progression


Green marker must remain partially visible during installation

Interior of the cup with very soft silicone disc to protect the fetal scalp

180° mobility of the frontal plane

Ergonomic handle

Variable diameter adapter for connection to vacuum cleaner

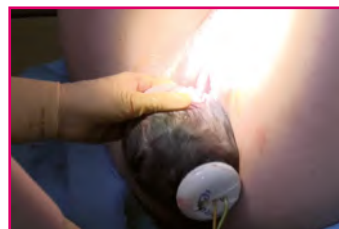

Icup²

Designed by Prof. Didier Riethmuller
CHRU Besançon
Modèle 01.451 - ø 5 cm.

Traceability ensured by 4 self-adhesive labels present on each packaging for the patient file. Industry's best adhesion, even on flat surfaces.

Ref. 46116

Packaging: box of 20 units in sterile individual packaging.




AMNIOTIC HOOK STERILE

Sterile amniotic perforator

Allows rupture of membranes: atraumatic for the cervix and fetal presentation, physiological curvature. Length 250mm. Sterilization with ethylene oxide. Composition: molded plastic Ref ABS without latex and PVC. Conformity to Directive 93/42/EEC (CE marking).

Ref. 76002

Packaging: box of 150 pieces. Sterile individual peel packaging

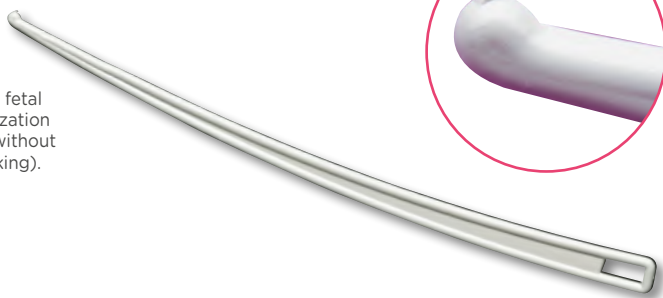

BABY CLAMP STERILE

APPLICATION :

- Safety and ergonomics for clamping the umbilical cord during childbirth



Bahr clamp

Material: polyamid. Sterilization with ethylene oxide. Conformity to Directive 93/42/EEC (CE marking). Length 54.10mm.

Ref. 76009

Packaging: box of 150 clamps. Individual peel packaging



CLAMP CUTTER STERILE

Clamp cutter

Plastic forceps used for removal of Bahr clamps. Length: 120mm. Sterilization with ethylene oxide. Compliance Directive 93/42/EEC (CE marking).

Ref. 47301

Packaging: box of 25 units. Individual peel packaging



CORD CARE KIT WITH TUBULAR ELASTIC BANDAGE

COMPOSITION :

- 1 blister with 3 compartments
- 5 gauze pads 10 x 10 cm 12 ply and 13 threads
- 1 baby clamp
- 1 tubular elastic bandage
- 1 pair of round-ended scissors

Cord care kit with tubular elastic bandage

No latex or PVC.

Ref. 76024

Packaging: box of 48 units



ABORTION SET

For enduterine aspiration, sterile single-use with male or female tip

ABORTION SET 3

Ø 12 MM

Mouthpiece,
tube and sleeve



TECHNICAL CHARACTERISTICS :

- Tube length 1.80 m.
- 1 revolving handle in styrenic resin
- 1 PVC suction tube
- 1 styrene resin connector
- Absence of phthalate.
- Compliance with Directive 93/42/CEE (CE marking)

Abortion set 3, Ø 12 mm

Mouthpiece, tube and sleeve

Ref. 46075

Packaging: carton of 20 units Individual sterile peelable sachet



ABORTION SET 2

Ø 12 MM

Tube
and sleeve



TECHNICAL CHARACTERISTICS :

- Internal diameter 12mm,
- Tube length 1.80 m.
- 1 turning handle in K-Resin
- 1 PVC suction tube, diameter 12 mm
- Compliance with Directive 93/42/CEE (CE marking)

Abortion set 2, Ø 12 mm

Tube & sleeve

Ref. 46074

Packaging: carton of 20 units Individual sterile peelable sachet



DISPOSABLE HANDLE Ø 12 MM



Disposable sleeve

Ref. 46076

Packaging: box of 20 units

Sterile individual peel pouch packaging



APPLICATION :

- Intermediate fenestrated part with movable ring that fits between the curette and the suction tube

TECHNICAL CHARACTERISTICS :

- Membership:
Handle, pressure ring: polystyrene, drain hose adapter: nylon, inner ring: silicone, latex and PVC free
- Compliance with Directive 93/42/CEE (CE marking)
- Gamma sterilization

CURETTE sterile for endo-uterine aspiration

FLEXIBLE

BENEFITS :

- Flexible cannula with a rounded end.
- 2 or Greenures (one on each side) to allow endo-uterine aspiration. The cannula is inserted into a conical tip which adapts perfectly to a rotating handle.
- The color coding of the conical tips makes it easier to identify the diameter of the curettes

TECHNICAL CHARACTERISTICS :


- Overall curette length 255mm (including connector).
- 7 diameters to choose from.
- Sterilization with ethylene oxide
- Composition Polyethylene
- Directive Compliance 93/42/EEC (CE marking)

Soft curette

Ø 4 mm, yellow. Ref. 48689 // Ø 5 mm, green. Ref. 48690
 Ø 6 mm, blue. Ref. 48691 // Ø 7 mm, black. Ref. 48692
 Ø 8 mm, red. Ref. 48693 // Ø 9 mm, white. Ref. 48694
 Ø 10 mm, orange. Ref. 48695 // Ø 12 mm, purple. Ref. 48697

Packaging: box of 12 boxes of 20 pieces

Sterile individual peel pouch packaging

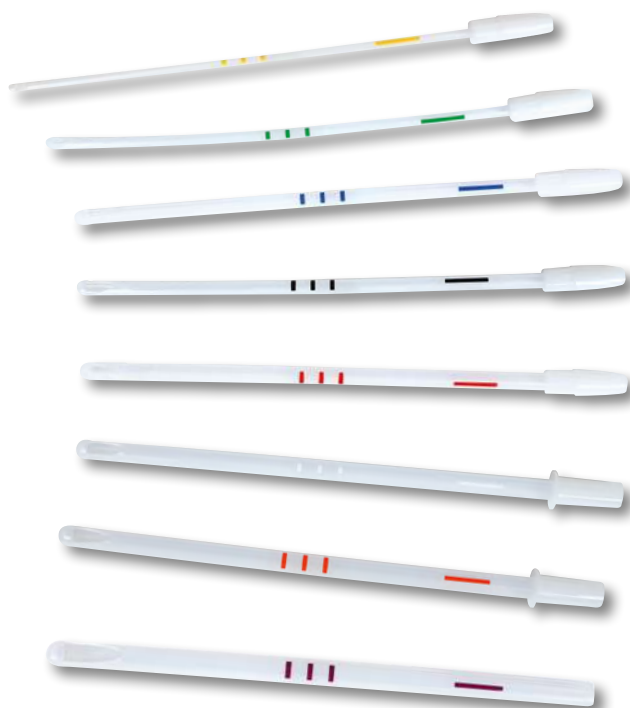

RIGID AND CURVE



TECHNICAL CHARACTERISTICS :

- Composition Kresin
- Length 190mm
- Sterilization with ethylene oxide
- Rounded end, slightly curved
- An opening for suction
- 7 different sizes
- Universal tip that adapts perfectly to a revolving handle
- Each diameter corresponds to the number of weeks of amenorrhea
- Compliance with Directive 93/42/CEE (CE marking)

Rigid and Curve Curette

Ø 6 mm. Ref. 47298 // Ø 7 mm. Ref. 47295
 Ø 8 mm. Ref. 47294 // Ø 9 mm. Ref. 47293
 Ø 10 mm. Ref. 47292 // Ø 11 mm. Ref. 47297
 Ø 12 mm. Ref. 47296

Packaging: box of 12 boxes of 20 pieces

Sterile individual peel pouch packaging



RIGID AND STRAIGHT



TECHNICAL CHARACTERISTICS :

- Composition Kresin
- Length 190mm
- Sterilization with ethylene oxide
- Rounded end
- An opening for suction
- 7 different sizes
- Universal tip that adapts perfectly to a revolving handle
- Each diameter corresponds to the number of weeks of amenorrhea
- Compliance with Directive 93/42/CEE (CE marking)

Rigid and Straight Curette

Ø 6 mm. Ref. 47201 // Ø 7 mm. Ref. 48698
 Ø 8 mm. Ref. 48699 // Ø 9 mm. Ref. 48700
 Ø 10 mm. Ref. 48701 // Ø 11 mm. Ref. 48702
 Ø 12 mm. Ref. 48703 // Ø 14 mm. Ref. 47205


Packaging: box of 12 boxes of 20 pieces

Sterile individual peel pouch packaging



PESSARY : THE ALTERNATIVE TO SURGERY

Gyneas Laboratories are the pessary specialists.

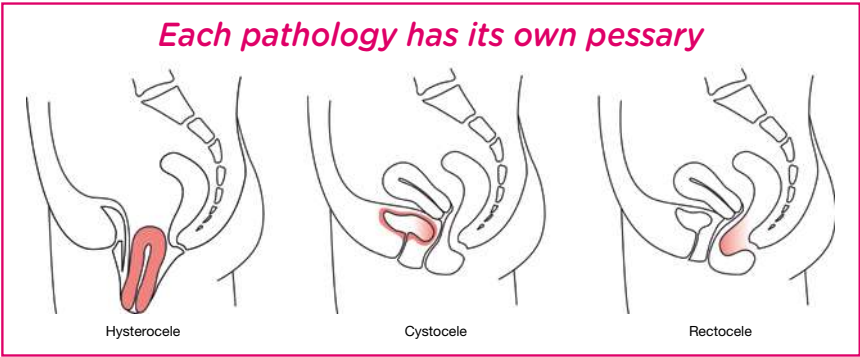

FOR WHO ?

Your patients with prolapse or stress urinary incontinence:

- in prevention for the sporty woman
- the young mother in need of transitional support
- postmenopausal women who can thus avoid or postpone surgery!

Choose the right pessary according to the pathology and morphology of your patient

Pathologies	Prolapsus 1-2°	Prolapsus 2-3°	Cystocele	Rectocele	Urinary incontinence	Gap from the collar	Sexual relations
Pessaries							
Cube	X	X	X	X	X		
Ring	X		X		X		X
Dish Style	X		X		X		X
Donut		X	X	X			

CUBE Gyn & Cube



Feminine cubic pessary, designed and developed by Gyneas Laboratories.
“ The most prescribed pessary by healthcare professionals ”

INDICATION :




- Stress urinary incontinence during sustained exercise (aerobics, tennis, jogging, aquabike, ...)
- Moderate or exteriorized prolapse
- Medium Cystocele and Rectocele
- After childbirth: prolapse or transient incontinence

TECHNICAL CHARACTERISTICS :

- Latex-free medical grade silicone, Kevlar wire.

BENEFITS :

- 3 sizes
- Standard size fits 80% of patients
- Trial kit available

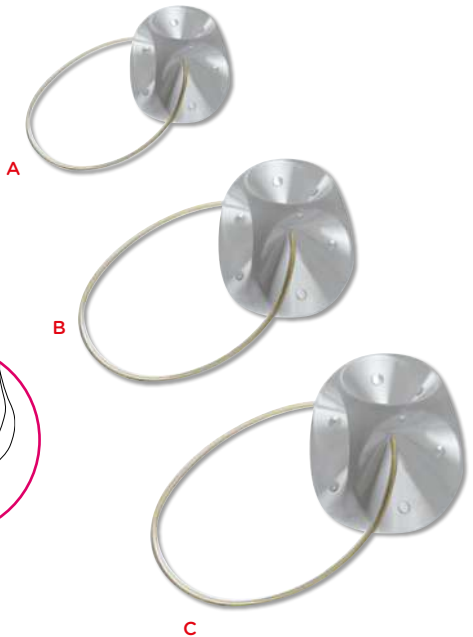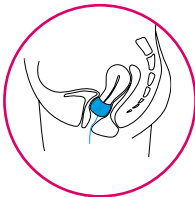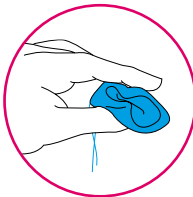

Gyn & Cube pessary

- A** Small Ø sup. 22 mm, Ø inf. 30 mm, h. 30 mm. - Ref. 47288.
- B** Standard Ø sup. 28 mm, Ø inf. 38 mm, h. 38 mm. - Ref. 47289.
- C** Large Ø sup. 32 mm, Ø inf. 44 mm, h. 44 mm. - Ref. 47290.

Packaging: individually



PESSARY **CUBIC**



*Choose the right pessary according
to the pathology and morphology
of your patient*

INDICATION :

- Stress urinary incontinence during exercise
- Moderate or exteriorized prolapse.
- Cystocele and rectocele.
- After childbirth: prolapse or transient incontinence.
- For women with a contraindication to surgical treatment.

TECHNICAL CHARACTERISTICS :

- Latex-free medical grade silicone.

BENEFITS :

- 11 sizes

INSERTION:

The recommended position is identical to that used for inserting a tampon: for example, standing with one foot on a chair or lying on your back with your head and shoulders well up on pillows. To facilitate installation, the pessary should be lubricated. It should be pressed between the thumb and forefinger, spread the lips with the fingers of the hand opposite side and insert it into the vagina, then push it inside with a finger in the direction of the inclination of the vagina.

REMOVAL:

Do not pull on the cord to remove the pessary. It is necessary, in priority, to eliminate the suction which is exerted by inserting the finger between the pessary and the vaginal wall, then compress the pessary between thumb and forefinger and gently withdraw it following the orientation of the vagina and using the cord as a guide. A lubricant facilitates the operation.

Cubic pessary


T0 - 25 mm. - Ref. 47879 // T1 - 29 mm. - Ref. 47880
T2 - 32 mm. - Ref. 47881 // T3 - 36 mm. - Ref. 47882
T4 - 40 mm. - Ref. 47883 // T5 - 45 mm. - Ref. 47884
T6 - 49 mm. - Ref. 47885 // T7 - 55 mm. - Ref. 47886
T8 - 63 mm. - Ref. 47887 // T9 - 69 mm. - Ref. 47888
T10 - 74 mm. - Ref. 47889

Packaging: individually

PESSARY

DONUT

Gyn & Donut

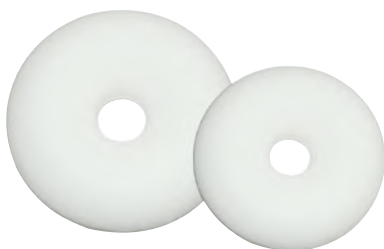

TECHNICAL CHARACTERISTICS :

- Hypoallergenic
- Made of medical grade silicone
- Available from size 51 to 95 mm

BENEFITS :

- 8 sizes
- Standard size fits 80% of patients
- Soft

INDICATION :

- 2nd and 3rd degree prolapse
- Cystocele
- Rectocele

Gyn & Donut pessary

T0 - 51 mm. - Ref. 47862. // T1 - 57 mm. - Ref. 47863.
T2 - 64 mm. - Ref. 47864. // T3 - 69 mm. - Ref. 47865.
T4 - 76 mm. - Ref. 47866. // T5 - 82 mm. - Ref. 47867.
T6 - 88 mm. - Ref. 47868. // T7 - 95 mm. - Ref. 47869.

Packaging: individually



Classe IIa

1304

DISH

Gyn & Dish



TECHNICAL CHARACTERISTICS :

- Hypoallergenic
- Soft medical grade silicone
- Size from 50 mm to 85 mm

BENEFITS :

- Available in 8 sizes
- Relatively thin ring circumference for optimal comfort
- Soft silicone for the well-being of the patient
- Elastic
- Prescribing

INDICATION :

- Mild 1st degree prolapse or cystocele
- Cystocele
- Rectocele

Gyn & Dish pessary

T0 - 50 mm. - Ref. 47854. // T1 - 55 mm. - Ref. 47855.
T2 - 60 mm. - Ref. 47856. // T3 - 63 mm. - Ref. 47857.
T4 - 68 mm. - Ref. 47858. // T5 - 73 mm. - Ref. 47859.
T6 - 80 mm. - Ref. 47860. // T7 - 85 mm. - Ref. 47861.

Packaging: individually



Classe IIa

1304

RING

Relatively thin ring circumference
for optimal comfort

Gyn & Ring



BENEFITS :

- Available in 9 sizes
- Flexible medical grade silicone, ensuring the well-being of the patient.
- Easily handled

INDICATION :

- Mild 1st degree prolapse or cystocele
- Cystocele
- Rectocele

Gyn & Ring pessary

Most popular sizes:

T1 - 51 mm. - Ref. 47870. // T2 - 57 mm. - Ref. 47871.
T3 - 62 mm. - Ref. 47872. // T4 - 70 mm. - Ref. 47873.
T5 - 74 mm. - Ref. 47874.

Other sizes:

T6 - 82 mm. - Ref. 47875. // T7 - 89 mm. - Ref. 47876.
T8 - 96 mm. - Ref. 47877. // T9 - 102 mm. - Ref. 47878.

Packaging: individually



Classe IIa

1304

PESSARY

CUBE



Dr.Arabin
dare to care

TECHNICAL CHARACTERISTICS :


- Hypoallergenic
- Made of medical grade silicone
- Available from size 25 to 75 mm

Cube pessary

T0 - 25 mm. - Ref. **50628** // T1 - 29 mm. - Ref. **50629**
T2 - 32 mm. - Ref. **50630** // T3 - 37 mm. - Ref. **50631**
T4 - 41 mm. - Ref. **50632** // T5 - 45 mm. - Ref. **50633**
T6 - 55 mm. - Ref. **50634** // T7 - 65 mm. - Ref. **50635**
T8 - 70 mm. - Ref. **50636** // T9 - 75 mm. - Ref. **50637**

Packaging: individually

The most
used sizes

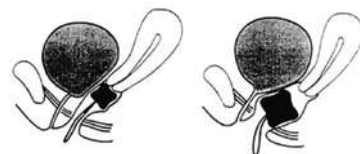

BENEFITS :

- 10 sizes
- Standard size fits 80% of patients
- Soft

INDICATION :

- 2nd and 3rd degree prolapse
- Cystocele
- Rectocele
- Urinary incontinence

GYNEAS
Pessary

DISH



Dr.Arabin
dare to care

TECHNICAL CHARACTERISTICS :


- Hypoallergenic
- Made of medical grade silicone
- Available from size 55 to 90 mm

Dish pessary

55 mm. - Ref. **50616**
60 mm. - Ref. **50617** // 65 mm. - Ref. **50618**
70 mm. - Ref. **50619** // 75 mm. - Ref. **50620**
80 mm. - Ref. **50621** // 85 mm. - Ref. **50622**
90 mm. - Ref. **50623**

Packaging: individually

The most
used sizes

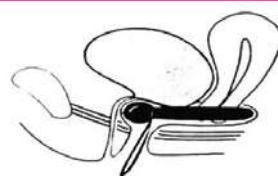

BENEFITS :

- 8 sizes
- Standard size fits 80% of patients
- Soft

INDICATION :

- 1st and 2nd degree prolapse
- Cystocele

CERCLAGE



Dr.Arabin
dare to care


TECHNICAL CHARACTERISTICS :

- Hypoallergenic
- Made of medical grade silicone

Cerclage pessary

ASQ 70/25/35 - Ref. **46756** // ASQ 65/17/32 - Ref. **46757**
ASQ 65/17/35 - Ref. **46758** // ASQ 65/21/32 - Ref. **46759**
ASQ 65/21/35 - Ref. **46760** // ASQ 65/25/32 - Ref. **46761**
ASQ 65/25/35 - Ref. **46762** // ASQ 65/30/35 - Ref. **46763**
ASQ 70/17/32 - Ref. **47117** // ASQ 70/17/35 - Ref. **47118**
ASQ 70/21/32 - Ref. **46764** // ASQ 70/21/35 - Ref. **46765**
ASQ 70/25/32 - Ref. **46766**

Packaging: individually

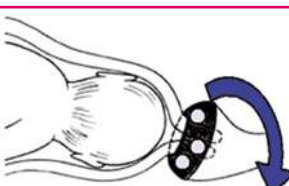

BENEFITS :

- 13 sizes according to the cervix of the patient
- Soft

INDICATION :

- Prevention of preterm birth

