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Thank you for your purchase of our product.

This product is a medical device used for examination in the ENT (ear, nose, throat) field.

This user manual explains the Frenzel Goggles S5 in detail so that its outstanding functions can be demonstrated fully. Before using this product, be sure to read this manual fully to ensure correct use.

After reading the manual, store it close by so that it can be retrieved whenever required.

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User Qualification

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This product is intended for use by a physician or healthcare provider under the supervision of a physician, and it is assumed that the user has received sufficient training in clinical procedures. Therefore, this user manual will not cover matters relating to clinical procedures. Details of clinical procedures should be assessed from a specialized viewpoint. Additionally, accountability for the use and maintenance of medical equipment lies with the user (hospital or clinic). It is crucial to fully understand the contents of this user manual before using the device in accordance with the regulations or directives applicable in the country of use.

All products manufactured and controlled by DAIICHI MEDICAL CO., LTD. for medical procedures are of the highest quality, and it is intended for non-invasive use by appropriately trained and licensed professionals only. To ensure maximum effectiveness and preserve its lifespan, this device must only be used for its intended purpose following a correct and acknowledged medical procedure. Only disinfectant agents recommended by the manufacturer should be used, and any guidelines for sterilization provided should not refer to any unvalidated process. Claims under the warranty are excluded, and DAIICHI MEDICAL CO., LTD. accepts no liability if device damage results from non-observance of these basic rules.

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This Instruction for Use (User Manual) is provided to provide information on the safe and appropriate use of the Frenzel Goggles S5. There have been no significant incidents related to this product. However, to ensure safe and effective use, please read and follow all instructions and warnings provided in this Instruction for Use. If you have any questions or concerns regarding this product, please contact the manufacturer or distributor.

If a significant incident occurs during the use of this product, please contact the following manufacturer or EC Representative.

However, please refer to the definition of a significant incident as described in this manual to determine whether the incident qualifies.

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Serious Incidents Qualifies

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The following events are considered serious incidents when they occur with this product:

- Cases where patients have suffered serious physical harm.
- Cases where there are problems with the product's functions, and there is a possibility that patients may suffer serious physical harm as a result.
- Cases where additional treatment is required for patients during product use that was not intended.
- Cases where there is a possibility of serious physical harm to patients due to improper use of the product.
- Other cases where serious issues have occurred due to product use.

If a significant incident occurs during the use of this product, please contact the following.

Manufacturer:

DAIICHI MEDICAL CO., LTD.

Head Office: 2-27-16, Hongo, Bunkyo-ku, Tokyo 113-0033 Japan

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This user manual uses the following warnings and indications. Be sure to observe the description provided with these warnings and indications when handling this product.

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Contraindications

The following specific situations are contraindicated for the use of the Frenzel Goggles S5 due to its potential harm to patients and users:

- Individuals with allergies to any of the materials used in the product, including PVC and artificial leather with Velcro.
- Individuals who have experienced allergic reactions to similar products in the past.
- Individuals with skin disorders or injuries in the area where the goggles will be placed.
- Individuals with systemic or localized infections in the area where the goggles will be placed.
- Pregnant or nursing individuals without prior approval from a physician.

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Warnings

- This product is not explosion-proof, so do not use flammable or explosive gases nearby.
- Use only the designated power source. If repair is required, it shall be our company or our designated agency or dealer, and no other person shall repair it.
- Never disassemble this product. Do not use equipment, such as medical electrical scalpels or mobile phones, that generate high frequency around this product while it is operating. It may cause malfunction.

- If the LED lights become hot, do not touch them inadvertently.
- Place flammable material away from this product. Failure to do so may cause a fire.
- Do not change a battery or operate switches with wet hands. It may cause electric shock.

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Prohibited Actions

- This product must be repaired by us or a designated distributor. It must not be repaired by anyone else.
- Do not use equipment, such as medical electrical scalpels or mobile phones, that generate high frequency around this product while it is operating. It may cause malfunction.
- Never place heavy objects on top of this product.

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Cautions

- Do not use this product beyond its rated service life. It may result in failure to light or damage during use.
- In all operations, do not exert excessive force or perform any sudden operations. It may cause accidents, malfunction, or damage.
- When performing operations, pay sufficient attention and care to the people and equipment around you. Failure to do may cause accidents, malfunction or damage.
- Minimize the use of disinfectant that is used for washing and cleaning. Failure to do so may cause deformation or damage to this product and related equipment.
- Do not drop this product, bump it against objects, subject it to excessive force, or damage it.
- When disposing of this unit, treat it as industrial waste. Be sure to ask an approved industrial waste disposal expert to dispose of the unit in accordance with the laws and regulations of the country of use.
- This product may release substances, including degradation products and leachable, which could have potential environmental impacts. Proper disposal of the product should be done by bringing it to a specific recycling point for electric and electronic equipment.

✖Otherwise, read the precautions described in the text fully.

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Precautions

- This product must be kept clean at all times since it is used mainly around patients' examination areas.
- After using the product, wipe off the face pad with an alcohol or alcohol swab and keep it clean. Note that the face pad is consumable. If a curing or crack is generated, replace it with a new one.
- When the medical examination is finished, remove any dirt and dust on the outer surfaces of the device. Dirt such as sebum and foundation adhere to the face pad.

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Side Effects:

When using the Frenzel Goggles S5, the following side effects may occur:

- (1) Prolonged contact with the goggles may lead to temporary skin irritation or itching. If you experience excessive discomfort, please discontinue the use of the device and consult a physician.
- (2) You may experience headaches during the use of the device. If the headache persists, please ensure the appropriate use of the device and consult a physician.
- (3) Prolonged use may result in a sensation of eye fatigue. Manage your usage time appropriately to prevent the accumulation of fatigue.

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Residual Risks:

- (1) Device performance may affect the accuracy of diagnoses. Therefore, if precise diagnosis is required, consider using other evaluation methods in conjunction.
- (2) For some medical conditions or disorders, this device may not be optimal. The final decision regarding diagnosis should be entrusted to a medical professional.

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Mitigation:

To minimize side effects and residual risks, please consider the following:

- (1) It is essential to follow the product's user manual for accurate usage.
- (2) If you experience uncomfortable symptoms during use, consult a physician immediately.
- (3) For certain medical conditions, other diagnostic methods or devices may be more suitable, so please consider the most appropriate diagnostic approach.

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Reporting and Contact Information:

In case of side effects or issues, please report them to the product manufacturer or the authorized distributor.

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DAIICHI MEDICAL instructs users of the Frenzel Goggles S5 to comply with the following:

- If the lenses become dirty with fingerprints, etc., wipe them off with a commercially available lens cleaner. Never use alcohol, benzine, thinner, or chemical solutions.
- Check the lens for scratches or dirt.
- Make sure the face pad is firmly attached to the body.
- Check that the Frenzel Goggles S5 can be fixed firmly to the face and that there is no damage to the main body, face pad, or fixing belt.
- If the face pad and the fixing belt are damaged or dirty, replace them with a new one.
- Check if the light-emitting diode lights up.
- Check if the battery level is sufficient. The brightness of the light-emitting diode decreases in proportion to the remaining battery power. If it becomes dark, replace it with a new battery.

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Regulations and Standards

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- The power source for this product must be connected to an emergency backup line in accordance with the regulations or directives that are applied in the country of use.
 - * This product falls under the category of controlled medical equipment of the Class category in accordance with regulations laid down in each country's domestic law.
- Observe all relevant regulations relating to the prevention of accidents.
- Tämän tuotteen virtalähteen on oltava kytketty hätävaravirtalinjaan käyttömaassa sovellettavien säännösten tai direktiivien mukaisesti.
 - *Tämä tuote kuuluu hallitun lääkintälaitteen luokkaan, noudattaen kunkin maan kotimaan lakia koskevia säännöksiä.
- Noudatettava kaikkia asiaan liittyviä sääntöjä liittyen onnettomuuksien ehkäisyyn.

EMC (Electromagnetic Compatibility) Related Items

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This unit is compliant with the EMC (electromagnetic compatibility) standard "IEC60601-1-2:2014" required for ensuring the safe use of medical electrical equipment.

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This unit is compliant with the safety standard "IEC60601-1:2005+AM1:2012" required for ensuring the safe use of medical electrical equipment.

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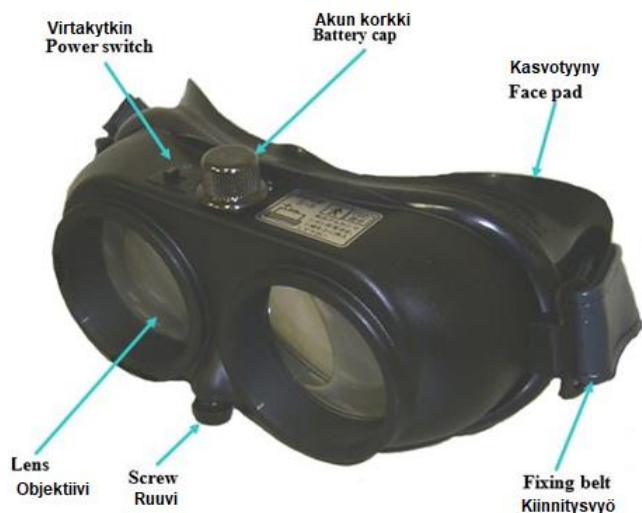
Note:

Use one AAA alkaline battery or manganese battery.

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Examination Procedure by Frenzel Goggles S5

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Caution

Remove the battery from the unit when the unit is not used for a long time.
 Otherwise, chemical substances may leak from the battery, damage the unit, and affect the human body.

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Warning

Insert the positive (+) and negative (-) batteries correctly.
 (It may cause failure or battery leakage.)

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Maintenance

Huolto

Cleaning appearance of the product

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- This product must be always kept clean since it is used around patient treatment areas. When medical treatment is finished, remove any dirt and dust on the outer surfaces of the main unit. If the outer surfaces of the main unit are dirty, moisten a cloth with neutral detergent diluted with water, wring the cloth and wipe off any dirt from the outer surfaces.

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- Dirt such as sebum and foundation adhere to the face pad. After using the product, wipe off the face pad with alcohol or alcohol swab and keep it clean. (Note that the face pad is consumable. If a curing or crack is generated, replace it with a new one.

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Cleaning lenses

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If the lenses get dirty by fingerprint etc., wipe them off with commercial lens cleaner.

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Caution

Never use alcohol, benzene, paint thinner, and drug solutions.

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Inspection before use

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Carry out an inspection before use definitely as follows.

- Check whether damage or dirt exists on the lenses.
- Check whether the battery power is sufficient.

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Caution

The brightness of the light-emitting diode gets weak in proportion to the battery power. If finding it gets dark, replace the battery with a new one.

- Check whether the light-emitting diode lights up.
- Check whether the face pad is attached to the goggles firmly.
- Check whether the goggles can be securely fixed to the face, and confirm that there is no damage to the goggles' main body, face pad, and fixing belt.
- Replace the fixing belt with a new one if it gets damaged or dirty.

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Environmental conditions (in use, transport and storage)

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Do not give a strong impact to the product during transport.

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Labelling

Tunnisteet

The device label information format adhered to by DAIICHI MEDICAL CO., LTD. is as follows:

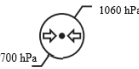
- Product name
- Model no.
- Voltage and current values
- Classification
- Intended Use
- UDI-DI number in barcode
- WARNING text
 - This product is not explosion-proof, so do not use flammable or explosive gases nearby.
 - Use only the designated power source. If repair is required, it shall be our company or our designated agency or dealer, and no other person shall repair it.
 - Never disassemble this product. Do not use equipment, such as medical electrical scalpels or mobile phones, that generate high frequency around this product while it is operating. It may cause malfunction.
 - If the LED lights become hot, do not touch them inadvertently.
 - Place flammable material away from this product. Failure to do so may cause a fire.
 - Do not change a battery or operate switches with wet hands. It may cause electric shock.
 - If the patient reports any abnormality or discomfort in the eyes or skin during use, discontinue use.

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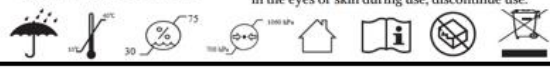
Label:
Tunnisteet:

MD	Classification: Class I Intended Use: For Nystagmus Examination	 - This product is not explosion-proof, so do not use flammable or explosive gases nearby. - Use only the designated power source. If repair is required, it shall be our company or our designated agency or dealer, and no other person shall repair it. - Never disassemble this product. Do not use equipment, such as medical electrical scalpels or mobile phones, that generate high frequency around this product while it is operating. It may cause malfunction. - If the LED lights become hot, do not touch them inadvertently. - Place flammable material away from this product. Failure to do so may cause a fire. - Do not change a battery or operate switches with wet hands. It may cause electric shock. - If the patient reports any abnormality or discomfort in the eyes or skin during use, discontinue use.
PRODUCT NAME Frenzel Goggles MODEL NAME S5 VOLTAGE DC1.5V CURRENT 0.17A		
SN	12345	MM/YYYY
	 (01) 04582368095424 (10) 12345	
		
 ENTFirst DAIICHI MEDICAL CO., LTD. 2-27-16, Hongo, Bunkyo-ku, Tokyo 113-0033 JAPAN tel: 81-3-3814-0111 fax: 81-3-3814-0135 email: first-en@first-med.co.jp web: www.first-med.co.jp/en		EC REP mdi Europa GmbH Langenhagener Str. 71 D-30855 Langenhagen, Germany

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DAIICHI MEDICAL CO., LTD. noudattaa seuraavaa laitteen tarratiedot muotoa: